

Before use, read the instructions for use.



MULTI-REG®

OCCLUSAL REGISTRATION MEDICAL DEVICE



MULTI-REG® is an occlusal registration and deprogramming device, developed by Prof. Stefano Pagano and Dr. Francesco Tortorella, useful in many dental fields whenever it is necessary to define, in a stable and reproducible way, a defined occlusal registration.

Light-curing material, available in pre-formed plates and can be light-cured with all UV or halogen light devices. The preformed plates are supplied in a light-proof box, reusable as a container for transport or storage of models etc.



Pieces: 20 | Taste: Mint | Color: Light Blue



STEP 1

Adapt the plate to the patient's lower arch after having carefully eliminated excess saliva, using your fingers to adapt the material to the teeth (preferably only on the occlusal area) trying not to press excessively.



STEP 2

Close the jaw, carry out the registration, and then proceed with the polymerization, initially in the molar region and then in the incisal area. Complete the polymerization of the entire arch in the vestibular direction, then reopen the jaw, repeat opening/closing a couple of times and then polymerize the lingual sides.



STEP 3

If necessary, refine the edges of the registration/plate obtained with a tungsten carbide bur and smooth with abrasive paper.



BART MEDICAL S.R.L.
Viale Vincenzo Randi 118
48124 Ravenna (RA) - Italy
+39 0544 523326
sales@bartmedical.com
www.bartmedical.com



Seguici su:

